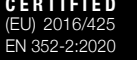
40327 Rev06-25

Rockets® [640001]
Rockets® Cord [640101]
Rockets® Full Detect [640901]
Comets® Cord [642101]
Twisters® Trio Cord [645101]
Twisters® Cord [644101]

WaveBand®2K [680001]
WaveBand®1K [681001]
Jazz Band®2 [670002]

made by

EN – User instruction Hearing protectors against harmful noise. Reusable hearing protectors. See instructions for use. **Warning** If the directions for use are not followed the protection provided by the ear defenders may be significantly compromised. Sudden or rapid removal of the earplugs from the ear canal could damage the eardrum. Wearing method for Jazz Band and WaveBand: under the chin banded earplugs. 1. Instructions for use • The earplugs must be inserted and maintained in accordance with the instructions for use (see Fig. 1-3 for headband instructions). • Check the earplugs are correctly fitted according to the instructions given on the box. • Ear defenders must be worn at all times in noisy environments. • Hearing protectors must be checked regularly for maintenance and cleaning requirements. Under the chin banded earplugs: only the headband can be cleaned as described below. The attached foam earplugs can be easily replaced with spare parts 6825 or 6830 if necessary. • The reusable earplugs and headbands can be cleaned with mild soap and water. • A mild disinfectant solution can be used. • After cleaning/disinfection, allow the hearing protectors to dry completely at room temperature. Storage: store in the original packaging in a clean, dry place. Maximum service life: as long as the product remains visually and hygienically flawless. **Warning for ear plugs with cord** The earplugs should not be worn if there is a risk of the connecting cord becoming entangled or caught during use. Do not remove the earplugs by pulling on the connecting cord. 2. This product may be adversely affected by certain chemical substances. Further information should be sought from the manufacturer. **Warning for banded earplugs** Harmful noise levels may be generated if the band is struck.

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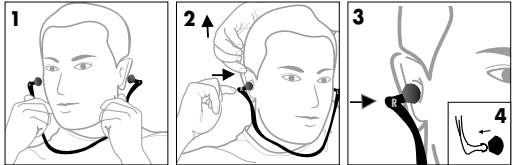
4 Replacement pods 6825/6830 - for use with Moldex bands only.

EN352-2:2020
PPE (EU) Regulation
2016/425

CE 0121

EAC TP TC 019/2011

EU type certification (Modul B):
SGS Fimko Ltd. (0598), Takomotie 8, FI-00380 Helsinki, Finland
Rockets®: Size 7–11 mm, Rockets® Full Detect: Size 7–11 mm, Twisters®: Size 10–14 mm, Twisters® Trio: Size 8–12 mm, WaveBand® 1K: Size: S/M/L, m: 12g, Material: POM
EU type certification (Modul B):
IFA (0121), Alte Heerstr. 111, 53757 St. Augustin, Germany
Comets®: Size 8–11 mm, Jazz-Band® 2: Size: S/M/L, m: 17g, Material: PC, TPE
EU type-examination has been issued by Notified body:
SATRA Technology Europe Ltd, Bracetown Business Park Clonee, Dublin, D15 YN2P, Ireland (Notified Body No.: 2777)
WaveBand® 2K: Size: S/M/L, Mass: 17g, Material: POM, TPE
Replacement Pods: Diameter 9–14 mm, Material: PU
Conformity assessment (Modul D):
IFA (0121), Alte Heerstr. 111, 53757 St. Augustin, Germany
Declaration of Conformity for these products can be found at:
www.moldex-europe.com/conformity

MOLDEx Moldex/Metric AG & Co. KG
www.moldex-europe.com

Head Office (DE):
Tübinger Straße 50
72141 Walldorfhäslach
Tel. +49 7127 8101-02
Fax +49 7127 8101-48